



LC-3DPrint Box

Specification	Value
Related voltage	110V/230V, 50/60Hz, 2,6A/1,3A
Power consumption	10W
Fuse	T2.0A, AC250V
Dimension	W41 XL44XH38 cm
Weight	22Kg

The LC-3DPrint Box is a revolutionary UV light box, suitable for post-curing 3D printing materials. The new LC-3DPrint Box is equipped with 12 UV light bulbs strategically placed inside the box. This ensures that a product is illuminated from all sides, which results in a quick and uniform curing cycle. The spacious interior (ø 26 cm, H 19,5 cm) allows you to easily cure multiple products at once. In addition, the box has enough space to place an articulator inside. If necessary, the UV light bulbs can be changed easily.

The new LC-3DPrint Box allows you to post-cure NextDent 3D materials. Post-curing is required in order to obtain the final material properties. After the printing process the printed material is not yet fully cured and should be treated. First, the material needs to be cleaned in an alcohol bath followed by the post-curing cycle. Post-curing is an UV light treatment to ensure that NextDent materials obtain full polymer conversion, through this the residual monomer is reduced to a minimum and the highest mechanical properties are obtained. This procedure is a necessary step to produce a biocompatible end-product. Always follow the instructions for use relevant to the corresponding material.